

Caution: A warning about your pulsers

In order to extend the life of your pulsers it is important to know that the life of the wires is directly related to how much flexing, pulling and bending your pulser wires receive. Although we work very hard to make the pulsers and the wires as durable as possible, there is no way to predict how long they will last. Anything that is connected with wires and handled on a regular basis, especially by stressed out people will eventually wear out. Proper use and storage can significantly increase their life.

Suggested holding positions for your Standard Pulsers



Suggested holding positions for your Mega Pulsers



Storage recommendations for your Pulsers

